## Foley Catheter Securement Device

**h** Description  
3400LFC Foley Catheter Securement Device

C

0086

D


K

LV05010 Rev 2

## INSTRUCTIONS

English

- Select the area for the placement of the Grip-Lok. Note: The catheter should be already inserted.
  - Prepare the skin according to the standard hospital protocol for dressing application. Skin prep or hair removal may be required on some patients for better adhesion.
  - Hold the Grip-Lok in position while removing the paper backing from one side, then the other, to secure on the skin surface.
  - Open and separate the top fabric straps.
  - Place the drainage lumen between the foam strip and the back of the fabric straps.
  - Position the catheter so that the Y-point is centered at the keyhole in the top fabric straps.
  - Secure one of the fabric straps over the drainage lumen and the other over the catheter below the Y-point.
  - Adjust the securement of the catheter by applying the strap attachments tighter or looser as needed.
- Direction: for added skin protection, use skin prep pad prior to application of stabilization device.  
Note: replace securement device if soiled or saturated in fluid.  
Note: use of an alcohol swab will ease removal of securement device from the skin.

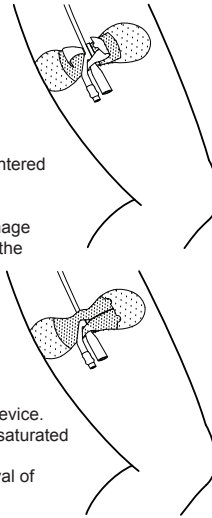This product does not contain any natural rubber latex.

This product is indicated for single patient use.

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